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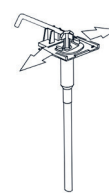
Abbildung/figure 5

Observe before mounting:

Unpack the dispenser from the box, remove the plastic packaging and ensure that the following parts are included (Fig.6):

- 1 x dispenser with disposable plastic pump (Fig. 5)
- 1 x operating lever, to be attached at the dispenser (Fig. 2)
- 1 x wall mounting plate (optional) (Fig.3), not necessary for use as a stand-alone unit
- 1 x drip tray (to be attached)
- 1 x bag with 3 screws/washers/dowels, as well as a metal strip
- 1 x user's manual, optional with disinfectant/soap labels

1 Installation/Operation**1.1 Dispenser assembly**

- The dispenser is delivered with drip tray and operating lever unassembled (Fig. 1).
- On the back side of the dispenser insert the operating lever and push it into place until an audible click is heard (Fig. 2); **Important: next insert the metal strip** (this serves to secure the operating lever in place and as an upper attachment point for the dispenser on the wall mounting plate).
- The dispenser can be used as a free-standing unit or wall-mounted unit.

1.2 Dispenser mounting (Fig.3)

A drill and a 6 mm drill bit as well as a screwdriver or cordless screwdriver are required for wall mounting. The wall mounting plate can be used as a drilling template. When replacing a dispenser, the existing wall mounting plate can be used. The dispenser can also be mounted directly on the wall, without the wall mounting plate.

- Select a suitable installation location and height. Recommended height: The top edge of the dispenser should be approx. 130 cm above the floor. Leave enough space under the dispenser so that the front part of the dispenser housing (possibly with the drip tray attached) can be fully opened for easy bottle replacement.
- Affix the wall mounting plate or dispenser body to the wall using the dowels and screws provided.
- Slide the dispenser over the wall mounting plate from above until you hear the locking pin click into place.

2 Set up

- To open the dispenser, press the push button under the operating lever (Fig. 2) and pull the housing cover forward.
- Slide the full bottle up over the dispenser pump from below and insert into the housing (fig. 4); pull the bottle holder (6) to the bottom of the bottle until it fits loosely. Close the housing cover, carefully guiding the pump nozzle through the opening at the top of the housing body; attach the drip tray.
- When using for the first time, press the operating lever down several times as far as it will go until liquid comes out of the pump nozzle; when using viscous gels, soaps or lotions the lever must

be pressed firmly up to 10 times before the solution will come out! In the event of problems, first use thin liquids (e.g. water), then viscous liquids.

3 Bottle or pump replacement and dosing

- Open the dispenser (Fig. 2), pull the empty bottle forward to remove, remove pump. Lock the pump on the wall and pull forward to remove. Dispose of the pump.
- Insert the new pump and continue as described in Step 2.
- The dosage pump is adjusted to the maximum flow rate of 1.8 ml per pump upon delivery.
- For use with viscous liquids, please only use the maximum dosage setting (dosing quantity 1.8 ml)!
- By moving the dosage fork to the right or left, the dispensing volume can be reduced to 1.2 ml or 0.8 ml per pump (Figure 5).

4 Dispenser cleaning/hygiene recommendations

- The dispenser is essentially maintenance-free, but its benefits can only be guaranteed with regular cleaning and disinfection of the dispenser housing and pump. Insufficient dispenser hygiene can lead to germ colonization of the dispenser system. Disinfectant or soap residues in the pump or on the nozzle can also cause problems. The dispenser should therefore be cleaned regularly in accordance with the recommendations for infection prevention and hygiene, and when necessary, also disinfected.

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- To do this, clean the housing with a mild cleaning agent only and, if required, treat with a surface disinfectant.
- It is possible to clean the housing in a suitable dishwasher.
- The **Robert Koch-Institute** expressly recommends the use of disposable bottles or hygiene containers for washing and care products. The German Medicines Act applies to hand sanitizers: empty bottles may only be refilled by a pharmacy under aseptic conditions.
- For hygienic reasons, the dispenser pump is designed to be a disposable pump, it should be changed at regular intervals when the bottle is exchanged, or at least every 3 - 4 months. It is not allowed to reprocess pumps, they must be replaced.