

Instruction for use AlproJet MixCup



Information on processing regarding the workflow and standard operation procedure (SOP) may be obtained at our website: www.alpro-medical.de, or can be requested by e-mail: info@alpro-medical.de, or by phone: **+49 7725 9392-0**.












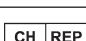



Contents

1. User instructions	4
2. Symbols used on product information, labels and instructions for use	4
3. Target group	4
4. Intended purpose – intended use	5
5. Application	5
6. Maintenance	6
7. Order data	6
8. Disposal instructions	7
9. Report of incidents	7

1. User instructions

Please attend to the following instructions in order to guarantee a trouble-free, economic and safe work.

2. Symbols used on product information, labels and instruction for use

	consult instructions for use
	caution, safety-related information
	catalogue number
	medical device
	unique device identifier
	serial number
	Health Industry Bar Code
	CE symbol for Medical Devices of class I
	UK Authorized Representative
	Swiss Authorized Representative
	date of manufacture
	manufacturer, Company ALPRO MEDICAL GMBH, St. Georgen
	company logo of ALPRO MEDICAL GMBH, St. Georgen

3. Target group

This instruction for use is intended for all users (doctors, dentists and professionals in charge of the processing of medical devices as well as hygiene professionals) in medical and dental facilities.

4. Intended purpose – intended use with the AlproJet series


AlproJet MixCup: For the preparation of AlproJet ready-for-use solutions for the maintenance and cleaning of dental aspiration systems.

AlproJet-D: Liquid concentrate for the daily disinfection and simultaneous cleaning of dental aspiration systems with and without amalgam separator.

AlproJet-DD: Liquid concentrate with enhanced disinfecting effect for the daily cleaning and maintenance of dental aspiration systems with and without amalgam separator.

AlproJet-W: Liquid concentrate for the weekly acid contra-cleaning and maintenance of dental aspiration systems with and without amalgam separator.

5. Application

 Use the AlproJet MixCup only with the AlproJet series!


AlproJet-D: 5 % application concentration

AlproJet-DD: 2 % application concentration

AlproJet-W: 5 % application concentration

2 L of a ready-to-use solution are required for one application. For this purpose dose 2 x **AlproJet** concentrate by using the dosage chamber of the dosage bottle (Fig. 1) and fill up in the **AlproJet MixCup** (Fig. 2) with drinking water (room temperature) to 2 L of a ready-to-use solution (Fig. 3). Afterwards aspirate 1 L of the ready-to-use solution through the aspiration tubes (Fig. 4) and pour the remaining 1 L into the spittoon bowl (Fig. 5).

Thereafter let take effect for at least 60 minutes and rinse the aspiration tubes and the spittoon bowl thoroughly with water just before starting work.

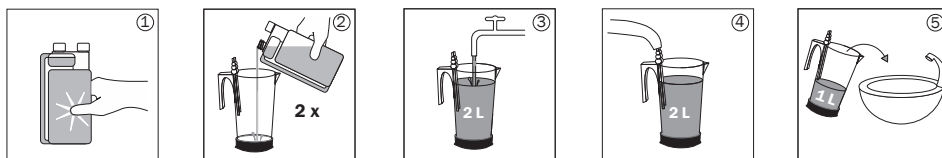

When using the AlproJet MixCup with AlproJet-D, AlproJet-DD or AlproJet-W, please follow the instructions on the respective product label.

Application only by qualified medical personnel with protective clothing.

6. Maintenance

Visible soiling can be removed with AlproJet-D or AlproJet-DD and a plastic brush.

7. Order data

AlproJet-D

REF 3104	1 L dosage bottle
REF 3100	10 L can
REF 3124	StarterSet



AlproJet-DD

REF 3120	1 L dosage bottle
REF 3115	5 L can
REF 3125-N	StarterSet



AlproJet-W

REF 3106	1 L dosage bottle
REF 3105	5 L can
REF 3108	10 L can



AlproJet MixCup

REF 3085	multifunctional container with adapter (double connector) for the maintenance and cleaning of aspiration system
REF 3088	multifunctional container with adapter (triple connector) for the maintenance and cleaning of aspiration system
REF 3090	connection adapter (double connector) for AlproJet MixCup
REF 3091	connection adapter (triple connector) for AlproJet MixCup



8. Disposal instructions

The AlproJet MixCup must be disposed of with the residual waste, the packaging can be recycled.

9. Report of incidents

Please report any serious incidents that may occur in connection with the product to:

ALPRO MEDICAL GMBH

E-Mail: quality@alpro-medical.de

Tel.: **+49 7725 9392-0**

and the national authority



Information on the preparation with regard to the workflow and all current work instructions, safety data sheets, instructions for use etc. can be found on our website:

www.alpro-medical.de

If you have any questions, please do not hesitate to contact us at **www.alpro-medical.de** or by phone **+49 7725 9392-0**.

You are also welcome to contact our sales representatives by phone or e-mail.