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GB
Order data AlproJet MixCup
REF 3085 multifunctional container with adapter (double connector) for the maintenance and cleaning of aspiration system
REF 3088 multifunctional container with adapter (triple connector) for the maintenance and cleaning of aspiration system
REF 3090 connection adapter (double connector) for AlproJet MixCup
REF 3091 connection adapter (triple connector) for AlproJet MixCup

Maintenance
Visible soiling can be removed with AlproJet-D or AlproJet-DD and a plastic brush.

Disposal instructions
The AlproJet MixCup must be disposed of with the residual waste, the packaging can be recycled.

Report of incidents
Please report any serious incidents that may occur in connection with the product to:
ALPRO MEDICAL GMBH
quality@alpro-medical.de
+49 7725 9392-0

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GB

User instructions
Please attend to the following instructions in order to guarantee a trouble-free, economic and safe work.

Target group
This instruction for use is intended for all users (doctors, dentists and professionals in charge of the processing of medical devices as well as hygiene professionals) in medical and dental facilities.

Intended purpose – intended use with the AlproJet series

AlproJet MixCup:
For the preparation of AlproJet ready-for-use solutions for the maintenance and cleaning of dental aspiration systems.

AlproJet-D:
Liquid concentrate for the daily disinfection and simultaneous cleaning of dental aspi-ration systems with and without amalgam seperator.

AlproJet-DD:
Liquid concentrate with enhanced disin-fecting effect for the daily cleaning and maintenance of dental aspiration systems with and without amalgam seperator.

AlproJet-W:
Liquid concentrate for the weekly acid contra-cleaning and maintenance of dental aspiration systems with and without amal-gam seperator.

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AlproJet-D:
application concentration
AlproJet-DD:
application concentration
AlproJet-W:
application concentration

2 L of a ready-to-use solution are required for one application. For this purpose dose 2 x **AlproJet** concentrate by using the dosa-ge chamber of the dosage bottle (Fig. 1) and fill up in the **AlproJet MixCup** (Fig. 2) with drinking water (room tempera-ture) to 2 L of a ready-to-use solution (Fig. 3). Afterwards aspirate 1 L of the ready-to-use solution through the aspiration tubes (Fig. 4) and pour the remaining 1 L into the spittoon bowl (Fig. 5). Thereafter let take effect for at least 60 minutes and rinse the aspiration tubes and the spittoon bowl thoroughly with water just before starting work.

When using the AlproJet MixCup with AlproJet-D, AlproJet-DD or AlproJet-W, please follow the instructions on the respective product label.

Application only by qualified medical personnel with protective clothing.

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