# Instruction for use AlproZyme

## AlproZyme

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# Contents

<b>1.</b> User instructions .....	<b>4</b>
<b>2.</b> Symbols used on product information, labels and instructions for use .....	<b>4</b>
<b>3.</b> Target group .....	<b>4</b>
<b>4.</b> Intended purpose – intended use .....	<b>4</b>
<b>5.</b> Application .....	<b>5–6</b>
<b>6.</b> Order data .....	<b>6</b>
<b>7.</b> Hazard and precautionary statements .....	<b>7</b>
<b>8.</b> Disposal instructions .....	<b>7</b>
<b>9.</b> Report of incidents .....	<b>7</b>

## 1. User instructions

Please attend to the following instructions in order to guarantee a trouble-free, economic and safe work.

## 2. Symbols used on product information, labels and instructions for use

	consult instructions for use
	use-by date YYYY-MM
	batch code (6 digits)
	catalogue number
	CE-symbol with identification number of the Notified Body (0123) for Medical Devices of class IIa, IIb or III
	manufacturer, ALPRO MEDICAL GMBH, St. Georgen
	company logo of ALPRO MEDICAL GMBH, St. Georgen

## 3. Target group

This instruction for use is intended for all users (doctors, dentists and professionals in charge of the processing of medical devices as well as hygiene professionals) in medical and dental facilities.

## 4. Intended purpose – intended use

Alkaline-enzymatic cleaning granulate for the pre-cleaning and pre-disinfection for the processing of medical and dental instruments, rotating precision instruments and endoscopes. Useable by soaking bath method as well as in ultrasonic devices and washer-disinfectors.

## 5. Application

### Pre-cleaning

- 5.1 Open the bag
- 5.2 Fill the tray (with removable sieve insert) with 2 L **warm** water (20 – 45 °C)
- 5.3 Mix thoroughly bag contents (1 bag) with warm water (warm water is only needed for mixing)
- 5.4 Soak soiled instruments completely and bubble-free
- 5.5 Observe indicated contact time (see table)

Efficacy	Concentration/%	Time/Minutes	
<b>Cleaning</b>	0.5	2 (at 45 °C)	in ultrasonic bath and soaking bath
<b>virucidal high level</b>	0.5 1.0	15 (at 40 °C) 60 (at 30 °C)	ultrasonic bath soaking bath (SOP)

### Further processing:

- 5.6 Remove the sieve insert with the instruments (if visibly soiled, repeat point 5.4 and 5.5 where required)
- 5.7 Rinse the instruments with suitable water (e. g. deionized, demineralized/drinking water to avoid lime residues)
- 5.8 For the comprehensive disinfection put the instruments in a suitable instrument disinfection solution (e. g. **BIB forte eco**) or in a washer-disinfector. In this case observe the contact times/hold times of the corresponding manufacturer!
- 5.9 After the contact time (according to manufacturer's instructions) rinse the instruments (manual preparation) with suitable water (see 5.7) and dry them
- 5.10 Check the integrity
- 5.11 According to risk assessment sterilize the instruments, if necessary wrapped

## 5. Application

### Important processing information:

A final disinfection must be carried out after use with **AlproZyme**.

**Note:** Prepare a fresh solution every working day (depending on degree of contamination even more often). Wear protective gloves, an apron impermeable to water and protective glasses!

## 6. Order data

### AlproZyme

REF	Description	
3753	Clinic package 100 bags (10 g each)	<p>CE 0123</p>

## Accessory

REF	Description	
4018	Immersion tray plastic, vol. 4 L for the reprocessing can be reprocessed up to 60°C in the washer-disinfector	

## 7. Hazard and precautionary statements

**Danger.** Causes severe skin burns and eye damage. May cause respiratory irritation. Can be corrosive to metals. Do not breathe dust. Wear protective gloves/protective clothing/eye protection/face protection.

**IF SWALLOWED:** Rinse mouth. Do NOT induce vomiting.

**IF ON SKIN (or hair):** Take off immediately all contaminated clothing. Rinse skin with water (or shower).

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

**Immediately call a POISON CENTER/doctor.**

## 8. Disposal instructions

Prepared solution may be disposed of highly diluted via waste water. The unused product respectively product residues must be disposed of as hazardous waste in compliance with the Directive 2008/98/EC on waste as well as national and regional regulations. If possible leave product in original container. Do not mix with other waste materials. Packaging contaminated with product is considered as hazardous waste and must be disposed of accordingly.

## 9. Report of incidents

Please report any incident that may occur in connection with the product to:

**ALPRO MEDICAL GMBH**

**and the national authority**

Mail: [quality@alpro-medical.de](mailto:quality@alpro-medical.de)

Phone: +49 7725 9392-0

Information on the preparation with regard to the work steps and all current work instructions, safety data sheets, instructions for use etc. can be found on our website: [www.alpro-medical.de](http://www.alpro-medical.de)

If you have any questions, please do not hesitate to contact us at **[www.alpro-medical.de](http://www.alpro-medical.de)** or by phone **+49 7725 9392-0**. You are also welcome to contact our sales representatives by phone or e-mail.