

ALPRO Wipes

Our wipes with and without alcohol for all areas of the dental practice:
CleanWipes, MinutenWipes, PlastiSept eco Wipes and MaxiWipes reel

CleanWipes

Alcohol-free wet wipes for the non protein fixing cleaning and decontamination of surfaces of medical devices and intact skin.

PlastiSept eco Wipes

Alcohol-free wet wipes for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines particularly with acrylic glass or metal surfaces.

MinutenWipes

Alcoholic wet wipes for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment.

MaxiWipes reel

Dry reel made of nonwoven for soaking with MinutenSpray-classic or PlastiSept eco for the disinfection and cleaning of surfaces of medical devices.



Application of CleanWipes




- non protein fixing
- free of aldehydes and phenols
- high quality tissue
(„Cotton touch“, 50 g/m²)
- pleasantly neutral fragrance
- free of oxidants
- dermatologically tested

CleanWipes are ready for use cleaning wipes, especially designed for the cleaning and decontamination of medical devices, hands and intact skin. Organic and inorganic residues are removed.


Decontamination:



of hands

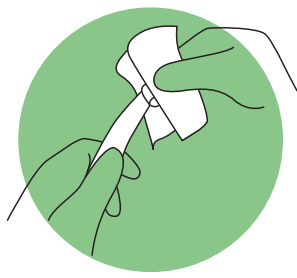

before reprocessing

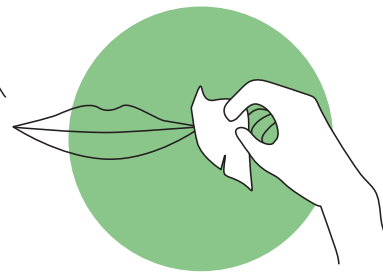

after impression taking



Application of PlastiSept eco Wipes



- free of aldehydes and phenols
- free of alcohol and QAC
- VAH certified – Surface disinfection with mechanics
- ready for use
- lint-free

PlastiSept eco Wipes are especially material compatible disinfection wipes for application on surfaces of non-invasive medical devices and medical inventory, in particular for surfaces sensitive to alcohol, surfaces of acrylic glass and for users who react sensitively to alcoholic quick disinfectants.

before reprocessing

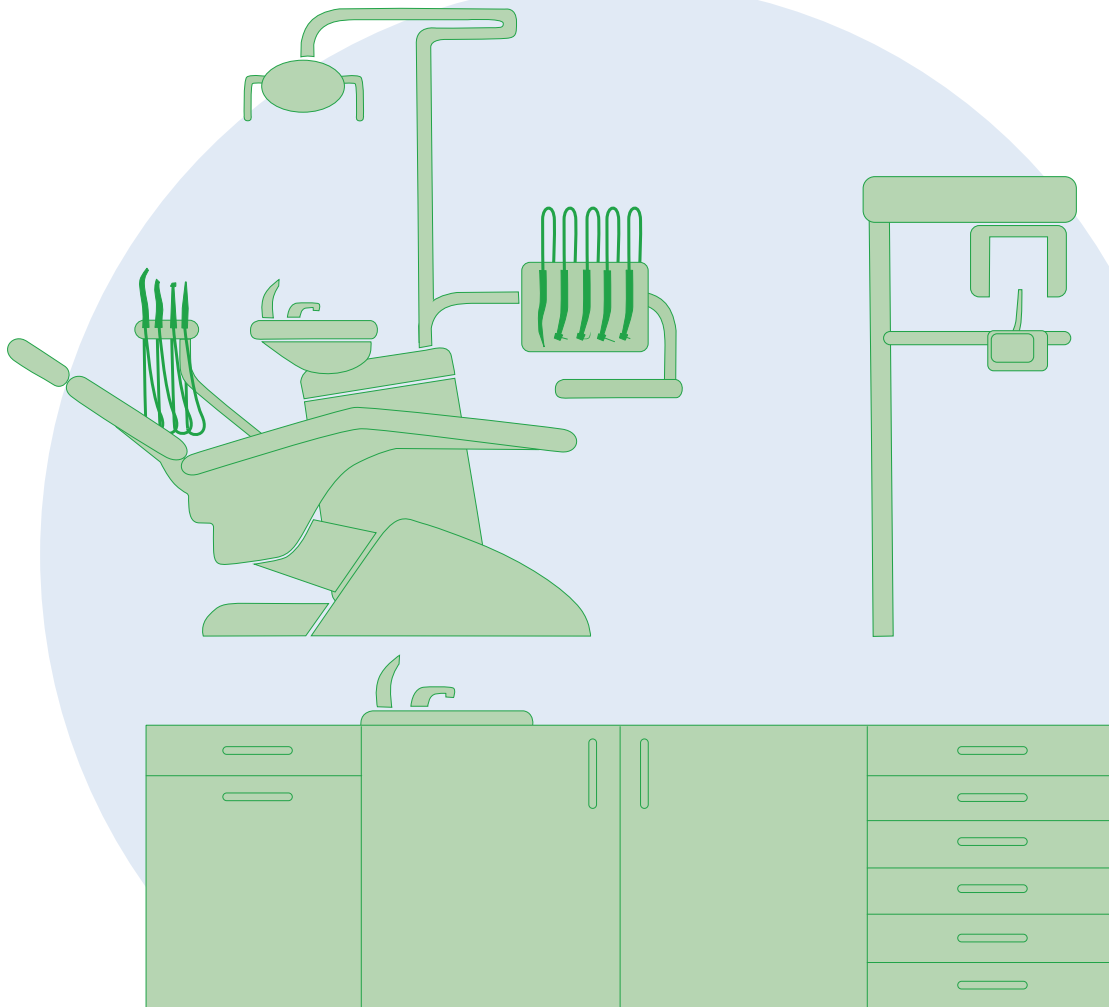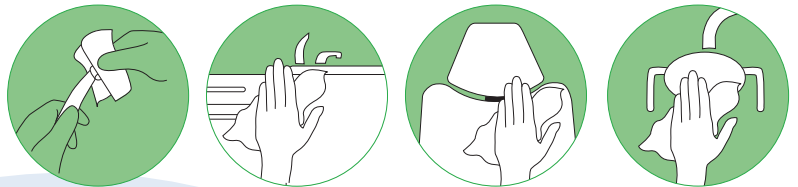

Application of MinutenWipes



Use only on alcohol-compatible surfaces!



- free of aldehydes and phenols
- VAH certified – Surface disinfection with mechanics under clean conditions
- very quick effect
- ready for use
- lint-free

MinutenWipes are suitable for the quick disinfection of surfaces of medical devices, suspension tables, unit grips, medical equipment, operating keys, hand pieces and contra-angles, turbines, etc.

Please observe the contact time!
Keep instruments and surfaces wet during the contact time.

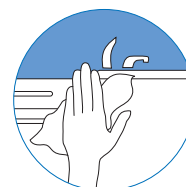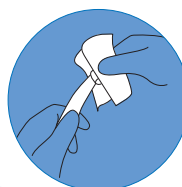



Application of MaxiWipes reel/L-reel

Attention: Please make sure that the dry disposable wipes to be soaked are compatible with the used disinfection solution!



- particularly high-quality and tearproof nonwoven cloths „cotton touch“ quality (50 g/m²)
 - extra large wipes (MaxiWipes reel: 17 x 26 cm, MaxiWipes-L reel: 20 x 40 cm), ideal for large surfaces
 - application with MinutenSpray-classic or PlastiSept eco
- 1 Dispose of the solution and wipes after the expiry date (1 month after soaking with active ingredient solution).
 - 2 Thoroughly wipe the inside of the dispenser box and lid with a cloth soaked with BC-San 100.
 - 3 Let the dispenser box and lid dry.
 - 4 Place the reel/L-reel bag into the dispenser box and cut open at the dividing line.
 - 5 Transfer the soaking solution used (MinutenSpray-classic/PlastiSept eco), the LOT number and the preparation date to the enclosed label and apply it visibly to the lid of the dispenser box.
 - 6 Put the opened bag over the edge of the bucket. Pull out a small piece of the innermost, pre-picked wipe from the middle.
 - 7 Slowly pour the ready for use solution (MinutenSpray-classic or PlastiSept eco) spirally from the middle outwards over the reel.
 - 8 Pull the innermost wipe of the reel through the star-shaped tear-off slot of the lid and dispose it. Put the lid on the dispenser and close until it locks.
 - 9 Let the wipes rest for 30 minutes.

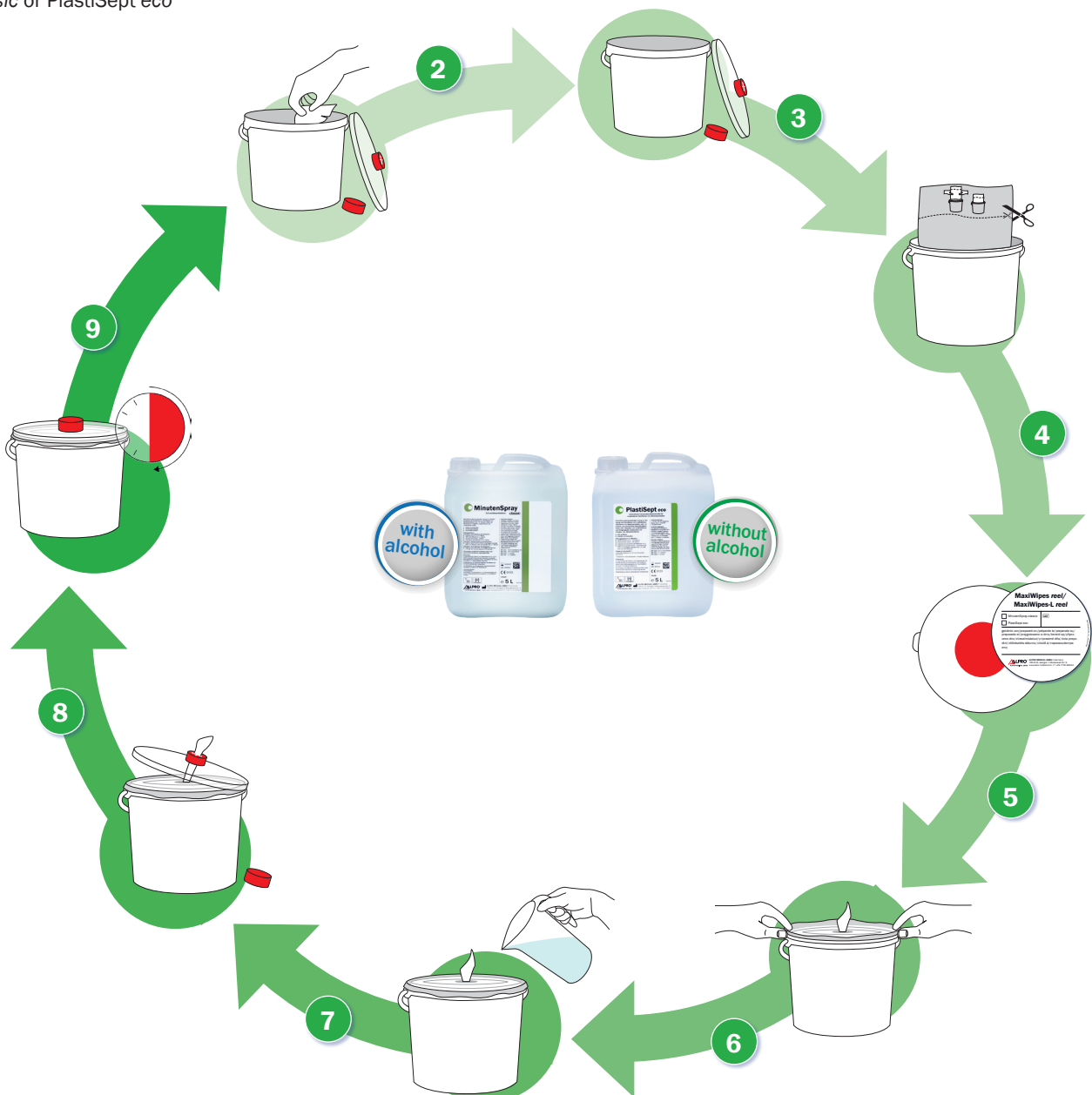

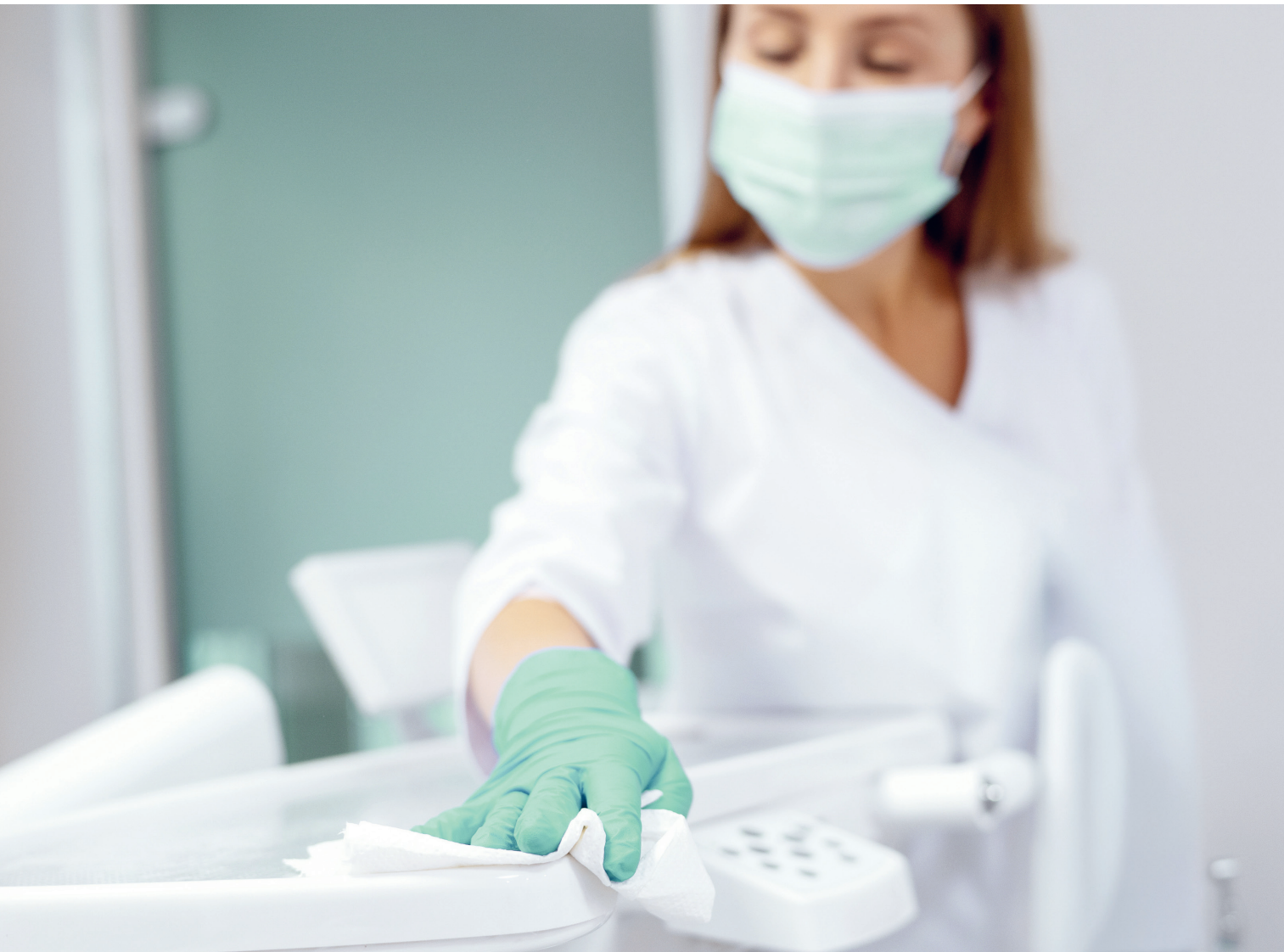

Our solutions for your entire practice hygiene



Our certified and eco-friendly products cover the entire spectrum of infection prevention in clinical practice — from hand, surface and instrument disinfection to the removal and prevention of biofilm in the procedural waterlines of dental treatment units. ALPRO is one of the world's leading manufacturers in the cleaning, disinfection, and maintenance of dental suction systems. Discover our extensive product range and services at:

www.alpro-medical.de





ALPRO MEDICAL GMBH
Mooswiesenstr. 9
78112 St. Georgen
Deutschland

Tel. +49 7725 9392-0
Fax +49 7725 9392-91
www.alpro-medical.de
info@alpro-medical.de