

# Standard Operating Procedure (SOP)

## Cleaning and comprehensive disinfection of not thermally reprocessable medical devices semicritical A

(e.g. light guides, polymerisation guides and optics of intraoral cameras)

WHAT	WHEN	HOW	BY MEANS OF
<p>Cleaning and disinfection of not thermally reprocessable medical devices</p> <p>(e.g. polymerisation lamps and intraoral cameras)</p>	<ul style="list-style-type: none"> <li>➤ After each patient</li> <li>➤ Before starting work</li> </ul>	<p>Wear protective gloves and protective glasses!</p> <ul style="list-style-type: none"> <li>➤ Cleaning: Thoroughly wipe contaminated surface from less contaminated to highly contaminated</li> <li>➤ Disinfection: Thoroughly disinfect cleaned surface with a soaked lint-free cloth</li> <li>➤ Recommendation: Clean (neutralize) <b>1 x per working day</b> (preferably in the evening) with tap water</li> </ul>	<ol style="list-style-type: none"> <li>1. Cleaning / disinfection with CleanWipes / PlastiSept eco / MinutenWipes Jumbo 50, ALPRO MEDICAL GmbH</li> <li>2. Comprehensive final disinfection with BC-San 100 ready for use (sodium hypochlorite) Contact time: 5 minutes</li> </ol> <p>Drinking Water</p>

**Instruction by:**

**Taken note of:**

\_\_\_\_\_  
Date/Signature (Practice operator)

\_\_\_\_\_  
Date/Signature (Staff)