

# Standard Operating Procedure (SOP)

## Cleaning and disinfection of contaminated surfaces after the treatment



WHAT	WHEN	HOW	WITH WHAT
Dental unit and contaminated surfaces in the treatment room	<ul style="list-style-type: none"> <li>➤ after each patient</li> <li>➤ prior to start of work</li> <li>➤ in case of visible contamination</li> </ul>	<p>wear protective gloves and protective glasses!</p> <ul style="list-style-type: none"> <li>➤ from slightly contaminated to highly contaminated</li> </ul> <p>e.g. lamp, doctor's element without transmission instruments, patient's chair, assistant's element, suction tubes, spittoon bowl, treatment cabinet, grips, external equipment (X-ray machines, prophylaxis devices etc.)</p>	<p>MinutenWipes Jumbo (2 Min.)</p> <p>PlastiSept-Wipes eco (5 min.)</p> <p>MaxiWipes reel* soaked with MinutenSpray-<i>classic</i> (2 min.) or PlastiSept eco (5 min.)</p> <p>MaxiWipes wet (5 min.) PlastiSept eco JumboWipes 50</p> <p>CidWipes eco (2 min.)</p>

\* Soak the dry wipes with 2.5 L MinutenSpray-*classic* or 1.6 L PlastiSept eco

Documentation of lot – cleaning and disinfection of the containers with BC-San 100 solution according to SOP for BC-San 100

Please note:

In case of changing the surface disinfection products it is recommended that before using the new disinfectant, all surfaces to be disinfected are thoroughly cleaned with a neutral cleaner (water and cleaner) and wiped off with clear water, to prevent possible reactions of remanence agents of the surface disinfectants which have been used before!

**Instruction by:**

**taken note of:**

---

Date/signature (practice operator)

---

Date/signature (staff)