

# Instruction for the processing of the wall dispenser for 1 L and 500 ml „Euro bottles”

## Step 1

The ALPRO wall dispenser can easily be cleaned and disinfected by the smooth surfaces e.g. with ALPRO MinutenWipes or PlastiSept-Wipes eco.

**For the processing of the pump system** at first pull off the drip tray and clean and disinfect it thoroughly.

Open the case.

To remove the pump we recommend using a suitable tool (e.g. a spatula)

Lift the pump lock with the tool and pull the pump simultaneously forward.



## Step 2

For the processing of the pump system prepare a ready for use solution e.g. with AlproZyme 1 % and place the pump in the solution. Pump 3 times until the solution is bubble-free inside the pump and let the solution take effect for 15 minutes.



Contact time:  
15 minutes

**For the extensive disinfection** place the pump afterwards in a 1 % solution of BIB forte eco, proceed as described above (wear personal protective equipment) and let the solution take effect for 30 minutes.



Contact time:  
30 minutes

## Step 3

**For the rinsing** place the pump for 30 minutes in drinking water and operate the pump at least 6 times to rinse the internal surfaces.



Contact time:  
30 minutes



## Step 4

Wipe the pump dry with a disposable wipe.

## Step 5

Assemble the dispenser in reverse order. Insert a new bottle of AlproSept-HDE/AlproSept-HDE gel or CREMANA®-wash and operate the pump handle 4-5 times to remove the residual water (from the rinsing) if necessary. The dispenser is now ready for further usage.



We recommend the cleaning and disinfection of the dispenser after every change of bottles at the latest.