

Standard Operating Procedure (SOP)

Cleaning and disinfection of aspiration systems (teeth cleaning + surgery)



WHAT	WHEN	HOW	BY MEANS OF
Cleaning of aspiration tubes and spittoon bowl drain	<p>prior to and/or after</p> <p>1) the use with sodium bicarbonate (e. g. airflow powder)</p> <p><u>or</u></p> <p>2) surgical treatments</p> <p>From Monday to Thursday</p> <p>For the contra-cleaning on Friday</p>	<p>Wear protective gloves + protective glasses!</p> <p>Preparation of 2 liters ready for use solution for the aspiration system:</p> <p>1) concentration 5 % / 10 %, i.e. 100/200 ml + 1900/1800 ml water (up to 2 ltr.)</p> <p>2) concentration 5 %, i. e. 50 ml + 950 ml water (up to 1 ltr.)</p> <p><u>First water, then concentrate</u></p> <ul style="list-style-type: none"> ➤ fill MixCup with warm water (40°C – 50 °C) ➤ measure the concentrate and add it to the water ➤ stir well <p>Preparation of 2 liters ready for use solution for the aspiration system :</p> <p>Alpro-Jet DD 2 %</p> <p>40 ml to 1960 ml warm water</p>	<p>AlproJet MixCup:</p> <p><u>Monday –Thursday:</u></p> <p><u>AlproJet-W (5-10%)</u></p> <p><u>Friday:</u></p> <p>Alpro-Jet-DD (2%) VAH certified</p> <p>(0,5% - 60 Min.</p> <p>1 % - 30 Min.)</p>

WHAT	WHEN	HOW	BY MEANS OF
cleaning and disinfection of the spittoon bowl	after each treatment if the patient has used it	Wear protective gloves + protective glasses! ➤ surface disinfectant ➤ spittoon bowl cleaner	MinutenSpray- <i>classic</i> PlastiSept <i>eco</i> MinutenWipes PlastiSept-Wipes <i>eco</i> AlproCleaner

Instruction by:

taken note of:

Practice operator

Staff