

Standard Operating Procedure (SOP)

Manual cleaning and disinfection of impressions and dental workpieces by immersion method



WHAT	WHEN	HOW	BY MEANS OF
<p>Cleaning and disinfection of the tray</p> <p>Labelling of the tray with: product name, indication of the volume of the tray in litres, date of the preparation, concentration</p>	<p>Prior to the new preparation of the cleaning and disinfection solution</p>	<p>Wear protective gloves, an apron impermeable to water + protective glasses!</p> <ul style="list-style-type: none"> ➤ empty the tray ➤ rinse the tray with water ➤ if necessary clean with a suitable brush ➤ wipe out the tray completely with a disinfection wipe 	<p>Plastic brush</p> <p>MinutenWipes (2 min.)</p> <p>MaxiWipes soaked with MinutenSpray-<i>classic</i> (2 min.)</p> <p>or with BC-San 100 solution</p>
<p>Preparation of the cleaning and disinfection solution for the impressions</p>	<ul style="list-style-type: none"> ➤ 2-3 days, if manufacturer confirmed the in-use stability by expertises ➤ at visible contamination (e. g. clouding, coagulation, streaking) 	<p>Wear protective gloves, an apron impermeable to water + protective glasses!</p> <p>Pour the cleaning and disinfection solution into the cleaned and disinfected tray</p> <ul style="list-style-type: none"> ➤ fill in solution ➤ mark the disinfection tray with the date of preparation (for longer period of use) 	<p>Suitable tray</p> <p>Cleaning and disinfection solution</p> <p>PrintoSept-ID (ready for use solution)</p>

WHAT	WHEN	HOW	BY MEANS OF
Appropriate pre-cleaning of the impressions, dental workpieces (e.g. prostheses, bridges etc.) for the disinfection	Immediately after removal from the mouth	Wear protective gloves, an apron impermeable to water + protective glasses! Contamination-protected transport to the sterilization area ➤ rinse impression, dental workpieces under running water	Running tap water
Disinfection of the impressions, dental workpieces (e.g. prostheses, bridges etc.)	➤ after the rinsing under running water ➤ prior to the transport to the laboratory ➤ after receipt from the laboratory	Wear protective gloves, an apron impermeable to water + protective glasses! ➤ bubble-free deposit ➤ impressions and dental workpieces to be covered completely with the solution ➤ contact time 5 min.	PrintoSept-ID (undiluted 5 min.) in-use stability 2 – 3 days depending on degree of soiling of the solution
Final step	➤ after the disinfection	Wear protective gloves, an apron impermeable to water + protective glasses! ➤ rinse under running water ➤ blow dry	running tap water compressed air (WL-Blow)

WHAT	WHEN	HOW	BY MEANS OF
Transport		Wear protective gloves, an apron impermeable to water + protective glasses! ➤ packed to the external laboratory ➤ direct processing in the own laboratory	

Instruction by:

taken note of:

Practice operator

Staff