

Standard Operating Procedure (SOP)

Manual cleaning of prostheses and dental workpieces in immersion bath or ultrasonic device



WHAT	WHEN	HOW	BY MEANS OF
Cleaning and disinfection of the tray	Prior to the new preparation of the cleaning solution	<p>Wear protective gloves, an apron impermeable to water + protective glasses!</p> <ul style="list-style-type: none"> ➤ empty the tray ➤ rinse the tray with water ➤ if necessary clean with a suitable brush ➤ wipe out the tray completely with a disinfection wipe 	<p>Plastic brush</p> <p>MinutenWipes, PlastiSept-Wipes eco</p> <p>MaxiWipes soaked with MinutenSpray-classic or PlastiSept eco</p>
Preparation of cleaning solution	<ul style="list-style-type: none"> ➤ 1 x per working day ➤ at visible contamination (e. g. clouding, coagulation, streaking) 	<p>Wear protective gloves, an apron impermeable to water + protective glasses!</p> <ul style="list-style-type: none"> ➤ fill in tap water ➤ add concentrate ➤ mix well 	<p>Suitable tray</p> <p>Cleaning solution</p> <ol style="list-style-type: none"> 1. warm tap water 2. TarClean 5 – 20 % depending on degree of soiling

WHAT	WHEN	HOW	BY MEANS OF
Pre-cleaning of dental workpieces (e. g. prostheses, bridges, dental splints, surgical masks etc.)	immediately after removal from the mouth	Wear protective gloves, an apron impermeable to water + protective glasses! During transport to the sterilization area to be protected against contamination ➤ rinse impression, dental workpieces under running water	Suitable water
Cleaning of dental workpieces (e. g. prostheses, bridges, dental splints, surgical masks etc.)	➤ prior to the transport to the laboratory ➤ after receipt from the laboratory	Wear protective gloves, an apron impermeable to water + protective glasses! ➤ bubble-free deposit ➤ impressions and dental workpieces to be covered completely with the solution ➤ contact time x min.	TarClean depending on degree of soiling 5 – 20 % in the tray max. 30 minutes, in the ultrasonic device approx. 5 minutes
Disinfection see work instruction „Manual cleaning and disinfection of instruments in immersion bath (wet/chemical)“	➤ after the cleaning ➤ prior to the transport to the laboratory ➤ after receipt from the laboratory	Wear protective gloves, an apron impermeable to water + protective glasses! ➤ bubble-free deposit ➤ impressions and dental workpieces to be covered completely with the solution ➤ contact time x min.	BIB-forte eco 0,5 % (60 min.) 3 % (30 min.) 4 % (5 min. ultrasonic d.) (tap water)

WHAT	WHEN	HOW	BY MEANS OF
Final step	After the disinfection	Wear protective gloves, an apron impermeable to water + protective glasses! ➤ rinse under running water ➤ blow dry	suitable water compressed air WL-Blow
Transport		Wear protective gloves, an apron impermeable to water + protective glasses! ➤ packed to the external laboratory ➤ direct processing in the own laboratory	

Instruction by:

taken note of:

Practice operator

Staff