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Reprocessing of the wall dispenser

Wall dispenser with lever arm for 1 L and 500 ml „Euro bottles”

Step 1

- The **ALPRO wall dispenser** can easily be cleaned and disinfected due to the smooth surfaces e.g. with **ALPRO MinutenWipes** or **PlastiSept eco Wipes**.
- **For the processing of the pump system** at first pull off the drip tray and clean and disinfect it thoroughly.
- Open the case.
- To remove the pump we recommend using a suitable tool (e.g. a spatula).
- Lift the pump lock with the tool and pull the pump simultaneously forward.

Step 2

- For the processing of the pump system prepare a ready for use solution e.g. with **AlproZyme** for virucidal disinfection 1 % and place the pump in the solution. Pump 3 times until the solution is bubble-free inside the pump and let the solution take effect for 15 minutes.
- **For the extensive disinfection** place the pump afterwards in a 1 % solution of **BIB forte eco**, proceed as described above (wear personal protective equipment) and let the solution take effect for 30 minutes.

Step 3

For the rinsing place the pump for 30 minutes in drinking water and operate the pump at least 6 times to rinse the internal surfaces.
Contact time: 30 minutes

Step 4

Wipe the pump dry with a disposable wipe.

Step 5

Assemble the dispenser in reverse order. Insert a new bottle of **AlproSept-HDE/AlproSept-HDE gel** or **CREMANA®-wash** and operate the pump handle 4 – 5 times to remove the residual water (from the rinsing) if necessary. The dispenser is now ready for further usage.

We recommend the cleaning and disinfection of the dispenser after every change of bottles at the latest.