


# Standard Operating Procedure (SOP)

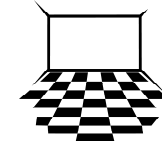

## Cleaning and disinfection of all contaminated washable surfaces

WHAT	WHEN	HOW	BY MEANS OF
Cleaning and disinfection of the two-chamber bucket	Prior to a new preparation of the cleaning and disinfection solution	Protective gloves <ul style="list-style-type: none"> <li>➤ Empty the bucket</li> <li>➤ Rinse the bucket with water</li> <li>➤ If necessary clean with a suitable brush</li> <li>➤ Wipe out the bucket completely with a disinfection wipe.</li> <li>➤ <b>Note: previously hygienic hand disinfection</b></li> </ul>	Plastic brush  MinutenWipes, PlastiSept eco Wipes  MaxiWipes soaked with MinutenSpray- <i>classic</i> , PlastiSept eco
Preparation of the cleaning and disinfection solution in the two-chamber bucket	<ul style="list-style-type: none"> <li>➤ Fresh preparation every workday</li> <li>➤ Change at visible contamination (e.g. clouding, coagulation, streaking)</li> </ul>	Protective gloves <ul style="list-style-type: none"> <li>➤ Fill only <b>tap water</b> into the first chamber</li> <li>➤ Measure lukewarm tap water for the solution and fill it into the second chamber</li> <li>➤ Add the concentrate to the second chamber</li> <li>➤ Stir well</li> </ul>	Suitable bucket  Cleaning and disinfection solution <ol style="list-style-type: none"> <li>1. Lukewarm tap water</li> <li>2. DesNet + 0,5 - 5 % depending on field of application</li> </ol> <p><b>See note on page 2</b></p>

WHAT	WHEN	HOW	BY MEANS OF
<p>Cleaning and disinfection of all contaminated surfaces in the practice</p> <p>Treatment room, processing room, laboratory etc.</p>	<p>Per working day after end of treatment and in case of visible contamination</p>	<p>Protective gloves</p> <ul style="list-style-type: none"> <li>➤ Soak MaxiWipes reel cloth with the solution of the second chamber and clean and disinfect the surfaces</li> <li>➤ Always wash and wring out MaxiWipes reel cloth in the first chamber before repeating the application</li> <li>➤ Moisten surfaces all over and let it take effect according to instructions (see below)</li> <li>➤ Disposal of the MaxiWipes reel cloth</li> </ul>	<p>Two-chamber bucket</p> <p>ALPRO MaxiWipes reel cloth</p> <p>Tap water</p> <p>DesNet +</p>

**Important information:**

**For general hygiene at a 0,5% dilution:**  
**5 L of concentrate yield 1000 L of ready for use solution**  
**0,5 % - 60 min.**  
**2,0 % - 15 min.**

**At a high risk of infection max. efficacy:**  
**4% - 60 min. or**  
**5% - 15 min. (e.g. critical areas as operating rooms)**

**Instruction by:**

**Taken note of:**

---

Date/signature (practice operator)

---

Date/signature (staff)