

Standard Operating Procedure (SOP)

Cleaning and comprehensive disinfection of not thermally reprocessable medical devices semicritical A

(e.g. light guides, polymerisation guides and optics for intraoral cameras)

WHAT	WHEN	HOW	BY MEANS OF
<p>Cleaning and disinfection of not thermally reprocessable medical devices</p> <p>(e.g. polymerisation lamps and intraoral cameras)</p>	<ul style="list-style-type: none"> ➤ After each patient ➤ Before starting work 	<p>Wear protective gloves and protective glasses!</p> <ul style="list-style-type: none"> ➤ Cleaning: Thoroughly wipe contaminated surface from less contaminated to highly contaminated ➤ Disinfection: Thoroughly sterilize cleaned surface with a soaked lint-free cloth ➤ Recommendation: Clean (neutralize) 1 x per working day (preferably in the evening) with tap water 	<p>1. Cleaning / disinfection with CleanWipes / PlastiSept eco / MinutenWipes Jumbo 50, ALPRO MEDICAL GmbH</p> <p>2. Comprehensive final disinfection with BC-San 100 ready for use 1 - 2 % active chlorine, sodium hypochlorite</p> <p>Contact time: 5 minutes</p> <p>Drinking Water</p>

Instruction by:

Taken note of:

Date/Signature (Practice operator)

Date/Signature (Staff)